### **FELIX-1: Negative Pressure Scavenger Kit**

#### **Intended Use:**

The FELIX-1 is a modified face-tent adapted to connect to hospital suction. This preassembled tool, compatible with Vapotherm's high velocity therapy, is intended to create a zone of relative negative pressure around the patient's nose and mouth, drawing in exhaled particles, which may mitigate the risk of transmission of potentially infectious particles while treating patients with airborne or droplet-transmitted infectious diseases, such as COVID-19. This is a single-use device.

1. Remove FELIX-1 from sealed pouch.
2. Ensure all connections are secured by pushing on all the connecting parts: (the adapter to the modified face-tent and tubing to adapter).
3. If using FELIX-1 with Vapotherm Precision Flow, size the appropriate Vapotherm cannula for the patient and place the cannula on the patient's face.
4. Place the modified face-tent on the patient's face.
  - a. Ensure the orientation is correct by placing the chin in the area identified as "CHIN."
5. Place strap over the patient's head and pull the strap until the modified face-tent is secure. Do not over-tighten the strap.
6. Connect all appropriate tubing to a suction canister and hospital vacuum system and initiate suction.
  - a. Vacuum should generally be set to the highest setting to ensure greatest flow from FELIX-1 or as clinically required.
7. If using FELIX-1 with Vapotherm Precision Flow, initiate Vapotherm High Velocity Therapy. Refer to Precision Flow® Instructions for Use.

#### **General Warnings:**

- FELIX-1 is designed to create a zone of negative pressure and flow, and is not intended for gas delivery.
- FELIX-1 is intended to be used with a filtered vacuum system.
- Set suction pressure according to appropriate clinical need.
- FELIX-1 is not intended to replace the need for Personal Protective Equipment (PPE) or negative pressure rooms.

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VAPOTHERM MAKES NO WARRANTY, EXPRESS, IMPLIED, STATUTORY OR OTHERWISE, WITH RESPECT TO THE PRODUCTS OR ANY OTHER ITEMS PROVIDED BY VAPOTHERM, AND HEREBY EXPRESSLY DISCLAIMS ANY OTHER FORM OF WARRANTY, INCLUDING, WITHOUT LIMITATION, ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

NOTWITHSTANDING ANYTHING ELSE IN THIS INSTRUCTIONS FOR USE OR OTHERWISE, TO THE EXTENT PERMITTED BY LAW VAPOTHERM SHALL NOT BE LIABLE WITH RESPECT TO ANY SUBJECT MATTER OF THIS INSTRUCTIONS FOR USE UNDER ANY CONTRACT, NEGLIGENCE, STRICT LIABILITY, OR OTHER LEGAL OR EQUITABLE THEORY, REGARDLESS OF WHETHER VAPOTHERM WAS ADVISED OF THE POSSIBILITY FOR SUCH DAMAGES, FOR ANY PUNITIVE, SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES (INCLUDING LOSS OF REVENUE OR LOST PROFIT) OR SIMILAR COMMERCIAL INDEMNITY.

**QTY** 1

For further information contact:

Vapotherm Inc  
100 Domain Drive  
Exeter, NH 03833  
USA  
Phone: 603-658-0011  
Fax: 603-658-0181  
[www.vapotherm.com](http://www.vapotherm.com)

May be patented  
[www.vapotherm.com/patents](http://www.vapotherm.com/patents)

Technical Support Line  
Domestic: 855-557-8276  
International: 603-658-5121  
[TS@vtherm.com](mailto:TS@vtherm.com)

AJW Technology Consulting GmbH  
Königsallee 106  
40215 Düsseldorf  
Germany  
Phone: +49 (0) 211 3013 2232