

This addendum introduces the ProSoft® Hi-VNI Nasal Cannulas series and is intended for the Appendices of the Precision Flow Hi-VNI®, Precision Flow® Plus, Precision Flow® Classic, and Precision Flow® Heliox Instructions for Use Manuals.

ProSoft Cannulas

Size	Part No.	Max Flow	Operational Flow Rates
Adult Long	ProSoft-XL	40	5-40 L/min (High Flow Cartridge)
Adult	ProSoft-ADULT	40	
Pediatric/Adult Small	ProSoft-PED-AS	40	
Pediatric Small	ProSoft-PED-S	20	1-20 L/min (High Flow Cartridge 5-20 L/min; Low Flow Cartridge 1-8 L/min)
Intermediate Infant	ProSoft-INT-INF	8	1-8 L/min (Low Flow Cartridge)
Infant	ProSoft-INF	8	
Premature	ProSoft- PRE	8	
Neonate	ProSoft-NEO	8	

ProSoft Hi-VNI Nasal Cannulas are labeled as single patient use only and must be replaced after 30 days. Disposable Patient Circuits, which includes the vapor transfer cartridge, disposable water path and delivery tube, are labeled as single patient use only and must be replaced by 30 days use on a single patient.

For the complete list of available cannulas and information on selection, application, and use refer to the “Connecting to the Patient” section of the appropriate Precision Flow manual.

English	For translated versions of this document, refer to the Vapotherm Website: https://vapotherm.com/international-documents
Italian	Per le versioni tradotte di questo documento, consultare il sito Web di Vapotherm: https://vapotherm.com/international-documents
German	Übersetzte Versionen dieses Dokuments finden Sie in der befinden sich auf der Vapotherm-Website: https://vapotherm.com/international-documents
French	Pour les versions traduites de ce document, visitez le site Web de Vapotherm : https://vapotherm.com/international-documents
Danish	Se oversatte versioner af dette dokument på Vapotherms websted: https://vapotherm.com/international-documents
Swedish	Översatta versioner av detta dokument finns på Vapotherms webbplats: https://vapotherm.com/international-documents
Norwegian	For oversatte versjoner av dette dokumentet, se Vapotherm-nettstedet: https://vapotherm.com/international-documents
Dutch	Vertalingen van dit document vindt u op de website van Vapotherm: https://vapotherm.com/international-documents
Bulgarian	За преведените версии на този документ вижте уебсайта на Vapotherm: https://vapotherm.com/international-documents
Romanian	Pentru versiunile traduse ale acestui document, consultați site-ul web Vapotherm: https://vapotherm.com/international-documents
Czech	Přeložené verze tohoto dokumentu najdete na webové stránce Vapotherm: https://vapotherm.com/international-documents
Slovak	Přeložené verzie tohto dokumentu nájdete na webovej stránke spoločnosti Vapotherm na adrese: https://vapotherm.com/international-documents
Portuguese	Para obter versões traduzidas deste documento, acesse ao site da Vapotherm em: https://vapotherm.com/international-documents
Spanish	Para obtener versiones traducidas de este documento, visite el sitio de web de Vapotherm: https://vapotherm.com/international-documents
Polish	Tłumaczenia tego dokumentu można znaleźć w witrynie internetowej firmy Vapotherm: https://vapotherm.com/international-documents