VAPOTHERM®
Aerosol-Adapter – Anweisungen

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42000158 Rev A (DE) Montiert in India

VAPOTHERM®
REF AAA-1

Aerosol Adapter
For use with Precision Flow®

QTY 10

NON STERILE

CH REP
AJW Technology Consulting GmbH
Kreuzplatz 2
8032 Zurich (Switzerland)
Telephone: +41 446 880 044

EC REP
AJW Technology Consulting GmbH
Breite Straße 3
40213 Düsseldorf (Germany)
Telephone: +49 211 54059 6030

Manufactured for:
Vapotherm, Inc.
100 Domain Drive
Exeter, NH 03833 USA
603-658-0011
May Be Patented:
www.vapotherm.com/patents

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42000158 Rev A Assembled in India

VAPOTHERM®
Aerosol Adapter - Instructions

Intended Use:
The Vapotherm Aerosol Adapter is intended to facilitate the connection between Vapotherm HVNI Technology and the Aergen® Aeroneb nebulizers.

Instructions for Use:
Place the Aerosol Adapter in line between the patient delivery tube and the nasal cannula or the Vapotherm TA-22 Tracheotomy Mask Adapter. For enhanced patient comfort consider a momentary pause of the HVNI Therapy when placing the adapter in the circuit then quickly ramping flow up to the desired level prior to starting the aerosol treatment.

Disconnect the patient interface from the delivery tube. Gently connect the adapter to the delivery tube to facilitate easy removal. Then, reconnect patient interface to the adapter. Once the Adapter is in place, start the aerosol treatment. It is important to maintain proper upright and horizontally level orientation of the adapter and nebulizer unit during the drug administration process.

WARNINGS:

- The adapter must be removed between aerosol treatments, as condensate will accumulate if left in the circuit for extended periods.
- When used for continuous aerosol treatments, condensate must be emptied at 30 to 60-minute intervals if not more frequently depending on rate of accumulation. The interval required is a function of the gas flow and drug delivery rates.
- Lower flows and higher drug delivery rates require more frequent condensate drainage.
- Failure to drain condensate may result in condensate delivery to the patient.

Refer to the Aergen PRO or SOLO Instructions for Use for aerosol treatment instructions. Refer to the Precision Flow Instructions for Use for operation of the Precision Flow®.

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