


| | | |
|---|--|--------------|
| Technical Bulletin | | |
|  | TITLE: HVT 2.0 Instructions for Use Updates | |
| DOC No.: TECH-00044_EN-GB | Revision: A | PAGE: 1 of 2 |

Distribution: Vapotherm website, Customer Service, Technical Support and Sales Team (Global)

Subject: This bulletin serves as an update to your HVT 2.0 Instructions for Use:

- 43000657 Rev. A (DE)
- 43000671 Rev. A (GB)
- 43000656 Rev. A (FR, IT, PT-BR, NL-BE, SV-SE, PT-PT, TR, ES-MX, ES, ID, JA-P)

See the referenced Instructions for Use listed above for complete instructions and all warnings and precautions.

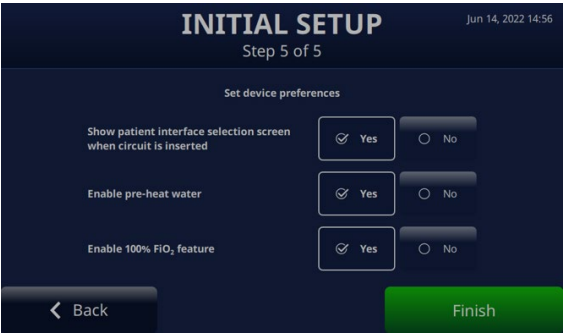
Date: August 2022

Please replace and/or add the following information to the sections as indicated:

HVT 2.0 Initial Device Setup Process


Step 5: Setup Device preferences

Replace *Figure 7: Device preferences settings* with the following:

| | |
|---|--|
| <p><b style="color: #4F81BD;">Step 5: Set up Device preferences</p> <p>Set up the device preferences.</p> |  <p style="text-align: center;"><i>Figure 7: Device preferences settings</i></p> |
|---|--|

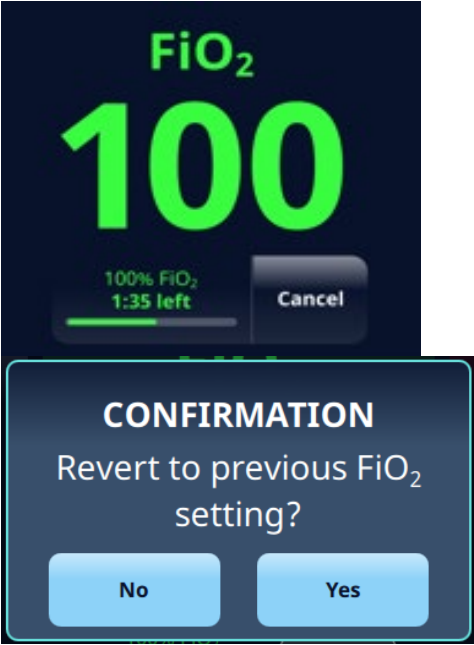
Select Patient Interface Type

6. Allow the DPC to prime with water before starting therapy. The device will indicate when priming is complete. It may take up to 5 minutes for the water supply to prime the DPC, including flexible, rigid or semi-rigid water bags and bottles.

| | | |
|--|--|--------------|
| Technical Bulletin | | |
|  VAPOTHERM | TITLE: HVT 2.0 Instructions for Use Updates | |
| DOC No.: TECH-00044_EN-GB | Revision: A | PAGE: 2 of 2 |

Select Therapy Parameters

Replace! Note and *Figure 31: 100% FiO2 timer display with the following:*

| | |
|--|--|
| <p>When the [100% FiO₂] button is pressed and confirmed, FiO₂ is administered at 100% for 2 minutes and a timer is displayed on the screen.</p> <p>When the 2 minutes end, the FiO₂ setting will return to the initial setting.</p> <p>! NOTE: To stop administering 100% FiO₂ before the 2 minutes has expired and revert to the previous FiO₂ setting, tap the [Cancel] button and <CONFIRM> by tapping [Yes]. To continue administering 100% FiO₂, Tap [No].</p> |  <p><i>Figure 31: 100% FiO₂ timer displays</i></p> |
|--|--|

How to transfer a patient

Add the following !Note to step 5:

! NOTE: When connected to AC power, charging of the transfer battery may be slowed or paused if operating the device outside the designed 20–25 degrees Celsius temperature range.

If you have any questions about this change, please contact Vapotherm Technical Support at ts@vtherm.com or get in touch with your local Vapotherm representative.