


<b>Technical Bulletin</b>		
		<b>TITLE: HVT 2.0 Instructions for Use Updates</b>
DOC #: TECH-00044	Revision: A	PAGE: 1 of 2

**Distribution:** Vapotherm website, Customer Service, Technical Support, and Sales Team (Global)

**Subject:** This Bulletin serves as an update to your HVT 2.0 Instructions for Use:

- 43000657 Rev. A (DE)
- 43000671 Rev. A (GB)
- 43000656 Rev. A (FR, IT, PT-BR, NL-BE, SV, PT, TR, ES-MX, ES, ID, JA)

See the referenced Instructions for Use listed above for complete instructions and all warnings and precautions.

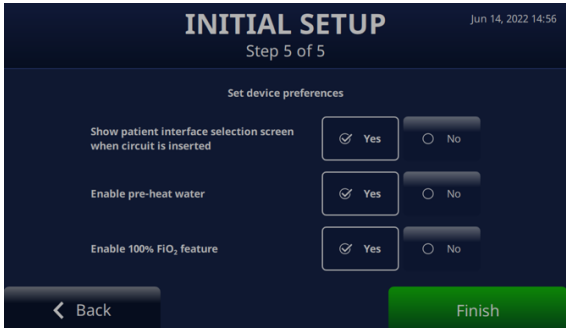
**Date:** August 2022

Please replace and/or add the following information to the sections as indicated:

## HVT 2.0 Initial Device Setup Process


### Step 5: Setup Device preferences

Replace *Figure 7: Device preferences settings* with the following:

<p><b>Step 5: Set up Device preferences</b></p> <p>Set up the device preferences.</p>	 <p><i>Figure 7: Device preferences settings</i></p>
---	--

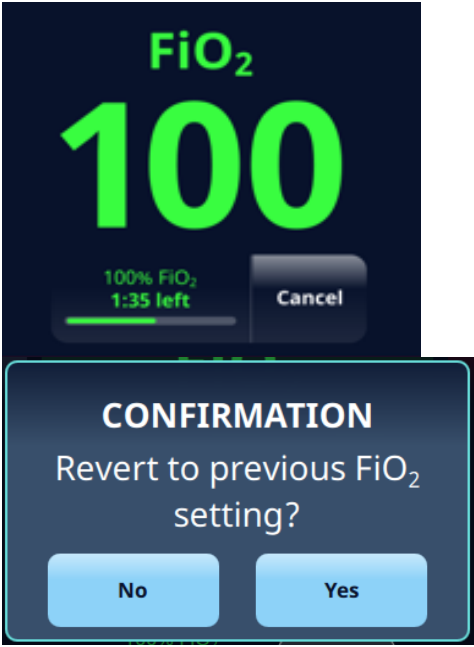
## Select Patient Interface Type

6. Allow the DPC to prime with water before starting therapy. The device will indicate when priming is complete. It may take up to 5 minutes for the water supply to prime the DPC, including flexible, rigid, or semi-rigid water bags and bottles.

<b>Technical Bulletin</b>		
 <b>VAPOTHERM</b>	<b>TITLE: HVT 2.0 Instructions for Use Updates</b>	
DOC #: TECH-00044	Revision: A	PAGE: 2 of 2

## Select Therapy Parameters

Replace !Note and *Figure 31: 100% FiO<sub>2</sub> timer display with the following:*

<p>When the [100% FiO<sub>2</sub>] button is pressed and confirmed, FiO<sub>2</sub> is administered at 100% for 2 minutes and a timer displays on the screen.</p> <p>When the 2 minutes expire, the FiO<sub>2</sub> setting returns to the initial setting.</p> <p><b>!NOTE:</b> To stop administering 100% FiO<sub>2</sub> before the 2 minutes expire and revert to the previous FiO<sub>2</sub> setting, Tap the [Cancel] button and &lt;CONFIRM&gt; by tapping [Yes]. To continue administering 100% FiO<sub>2</sub>, Tap [No].</p>	 <p><i>Figure 31: 100% FiO<sub>2</sub> timer displays</i></p>
---	--

## How to Transfer a Patient

Add the following !Note to step 5:

**! NOTE:** When connected to AC power, charging of the transfer battery may be slowed or paused if operating the device outside of 20 – 25 degrees Celsius temperature range.

If you have any questions about this change, please contact Vapotherm Technical Support at [ts@vtherm.com](mailto:ts@vtherm.com) or your local Vapotherm representative.